



**NOBIAK**  
 Υγραντήρας 250ml με Βαλβίδα Ασφαλείας 0.96 bar  
 Humidifier 250ml with Safety Valve 14PSI  
 code 0806205

**Instruction of use / Οδηγίες Χρήσης**

- 1. Fill humidifier up to the maximum fill line with water. Do not allow water to fall below the maximum fill line.
- 2. Attach the humidifier lid to the jar.
- 3. Attach the wing nut on the humidifier to oxygen source.
- 4. Test the humidifier for leaks by reclosing the humidifier outlet. The safety valve should activate. If the valve is not actuated or if any appreciable leakage occurs, check to make sure the humidifier lid is secure, is attached to the jar and the fitting to oxygen source is securely tightened.
- 5. Connect oxygen tubing (not included) to the nipple on the humidifier lid, be sure all connections are secure.
- 6. Turn on the oxygen source to the prescribed LPM.
- 7. The safety valve will alarm if there is a block or blockage in the delivery system.
- 8. Γιατί να ασφαλίσει με μίσηρη γρήγορα, με νερό. Μην αφήσετε το νερό να πέσει κάτω από την ελάχιστη γραμμή πλήρωσης.
- 9. Συνδέστε το σωλήνα του οξυγόνου στο δοχείο.
- 10. Επικεντρώστε το δοχείο του οξυγόνου στην προβλεπόμενη ποσότητα.
- 11. Ελέγξτε ότι η βαλβίδα ασφαλείας λειτουργεί κατάλληλα, κλείνοντας την είσοδο του υγραντήρα. Η βαλβίδα ασφαλείας πρέπει να ενεργοποιείται με ήχο και να μην υπάρχει διαρροή οξυγόνου. Εάν η βαλβίδα ασφαλείας δεν ενεργοποιείται ή αν υπάρχει διαρροή οξυγόνου, ελέγξτε για να βεβαιωθείτε ότι ο σωλήνας του οξυγόνου είναι σωστά στεγανωμένος και ότι η βαλβίδα ασφαλείας είναι σωστά στεγανωμένη.
- 12. Συνδέστε το σωλήνα του οξυγόνου στο δοχείο.
- 13. Επικεντρώστε την προβλεπόμενη ποσότητα οξυγόνου.
- 14. Η βαλβίδα ασφαλείας θα ελαττώνει τον κίνηση του νερού στο σωλήνα.

**Caution / Προσοχή**

- 1. The humidifier is equipped with a safety valve. When the safety valve sounds, check the downstream supply tubing and the oxygen device, remove any obstruction before resuming use. The flow is to proceed only after qualified personnel has been notified.
- 2. The water must be changed every 8 hours of use. It is recommended to use distilled water.
- 3. It is important to clean the humidifier regularly. Clean the humidifier with distilled water.
- 4. It is important to use distilled water in the humidifier. Do not use tap water.
- 5. It is important to use distilled water in the humidifier. Do not use tap water.
- 6. It is important to use distilled water in the humidifier. Do not use tap water.

QTY 1 piece LOT 20210326021 2021-05 2024-05

Distributed by: MEDICAL S.A. 28218 63222  
 Manufactured by: NOBIAK S.p.A. 2021-05  
 CE 2460  
 Rev. 02/07/19

